

– Instructions for Use –

Product Description: Vitále® Silicone Cover Dressings & Composite Dressings are absorbent, have a non-traumatic silicone border, and provide an ideal cover for wounds of many types. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated as a secondary dressing to cover primary treatments & dressings, or as a primary dressing on minor wounds, skin tears, etc. They assist in protecting the wound and preventing bacterial contamination or maceration.

Vitale Silicone Cover Dressings & Composite Dressings have an absorbent non-adherent pad in the center, held firmly in place by the silicone, uniquely formed in an oval shape open to the passage of fluids into the pad. The silicone adhesive allows easy & safe removal of the dressing at change time, without trauma to the patient. The soft and strong backing is made from flexible polyurethane that provides a waterproof barrier. This breathable film allows gases, including water vapor, to escape the dressing.

Vitale Silicone Cover Dressings & Composite Dressings are available in a selection of convenient sizes. They are sterilized by ethylene oxide and must not be resterilized.

Indications: Vitále Silicone Cover Dressings & Composite Dressings are for single use only and can be used to cover primary dressings and protect many types of wounds:



Superficial or Deep




Minimal to Heavy Exuding Wounds

- Pressure Sores
- Arterial Ulcers
- Venous Ulcers
- Diabetic Ulcers
- Post-Operative Wounds
- Skin Donor Site
- Second Degree Burns
- Post-Surgery Sites
- Minor Wounds and Lacerations


Side Effects/Interactions: None have been observed to date.

Storage: Vitále Silicone Cover Dressings & Composite Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.


Application:

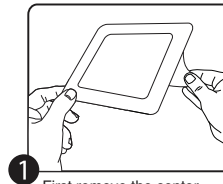

1 First remove the center backing film (or one side for the two smaller sizes).

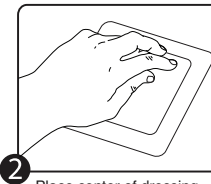

2 Place center of dressing over area and adhere one side.

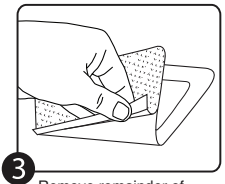

3 Remove remainder of backing and adhere.

– Product Line –

Catalog Number:	Description:	Dressing Size:	Pad Size:	Dressings/Box:
24203	Vitale Silicone Composite Dressings	2" x 3" (5cm x 7.5cm)	1" x 1.6" (2.5cm x 4cm)	30
24204	Vitale Silicone Composite Dressings	3.5" x 4" (9cm x 10cm)	2" x 2.4" (5cm x 6cm)	30
24206	Vitale Silicone Composite Dressings	5" x 6" (12.5cm x 15cm)	3" x 4" (7.5cm x 10cm)	30
24207	Vitale Silicone Composite Dressings	6" x 7" (15cm x 17.5cm)	4" x 5" (10cm x 12.5cm)	30
24209	Vitale Silicone Composite Dressings	9" x 9" (23cm x 23cm)	6.9" x 6.9" (17.5cm x 17.5cm)	15
24704	Vitale Silicone Cover Dressings	3.5" x 4" (9cm x 10cm)	2" x 2.4" (5cm x 6cm)	30
24707	Vitale Silicone Cover Dressings	6" x 7" (15cm x 17.5cm)	4" x 5" (10cm x 12.5cm)	30



CellEra, LLC Cincinnati, OH
www.cellerallc.com

STERILE EO Sterilized by exposure to Ethylene Oxide

Single-Use

Storage Temperature
100°F / 40°C
40°F / 5°C

Not Made with Natural Rubber Latex

Keep Dry