

# VITÁLE® Xeroform Petrolatum Dressings

## – Instructions for Use –

**Product Description:** Vitále® Xeroform Petrolatum Dressings are non-adherent occlusive dressings made of fine mesh gauze impregnated with 3% Xeroform (Bismuth Tribromophenate) in petrolatum. Xeroform Dressings are sterile, single-use devices.

### Indications:

Xeroform Petrolatum Dressings are intended for use as a primary contact layer to dress wounds such as lacerations, skin graft sites, newly sutured wounds, abrasions and minor or partial thickness burns. They may also be used as an initial layer in dressing pressure wounds with light exudate.

 **Superficial to Partial Thickness**

- Incision Sites
- Donor Sites & Skin Graft sites
- Abrasions & Minor Burns
- A Primary Occlusive Layer

 **Dry to Light Exuding**

### Contraindications:

1. Do not use on patients with hypersensitivity to Bismuth Tribromohenate.
2. Caution should be used when there is a history of atopy.
3. Do not use on heavily draining wounds.

### Warnings:

1. Do not use if unit package is opened or damaged.
2. Avoid multiple layers which may lead to maceration.
3. Discontinue use if irritation develops.
4. Avoid excessive packing and resultant pressure on the wound.

### Adverse Reactions:

Irritation, though rare, may develop in atopic patients or in patients sensitive to Bismuth Tribromophenate.

### Directions:

1. Clean and rinse the wound, using appropriate wound cleanser.
2. Before applying, skin around the wound should be clean and dry. Use a skin barrier prep as needed.
3. Using aseptic technique, remove dressing from the package and apply to moist wound bed.
4. Cover with a secondary absorbent dressing, and secure as appropriate.
5. Dressing change as directed, when strike-through occurs, or at least every 7 days.

## – Product Line –

Catalog Number:	Description:	Size:	Dressings/Box:
57108	Vitále Xeroform Petrolatum Dressings	1" x 8" (2.5 cm x 20 cm)	50
57202	Vitále Xeroform Petrolatum Dressings	2" x 2" (5 cm x 5 cm)	25
57404	Vitále Xeroform Petrolatum Dressings	4" x 4" (10 cm x 10 cm)	25
57509	Vitále Xeroform Petrolatum Dressings	5" x 9" (12.5 cm x 22.5 cm)	50



CellEra, LLC Cincinnati, OH  
www.celleralc.com



Do not use if package is damaged



Single-Use



Keep Dry



Storage Temperature

STERILE

Sterilized using Gamma Radiation

Rx Only



Not made with natural rubber latex

CCN:57xxxIFUEng03-24-22.0