

– Instructions for Use –

Product Description: Vitále® Collagen Powder Dressings are absorbent wound fillers made of type 1 microfibrillar collagen for the management of minimally to heavily exuding acute and chronic wounds and control of minor bleeding. Vitále Collagen Powder Dressing forms a gel on contact with exudate that provides a moist, physiologically favorable environment to support wound healing. Vitále Collagen Powder Dressing is applied directly to the wound to act as a wound filler and to absorb exudate and exudate components that may impair wound healing.

Vitále Collagen Powder Dressings are sterilized by exposure to gamma radiation and must not be re-sterilized. The dressing is an off-white loose powder packaged in convenient single-use, sterile, peel open packets, containing 1 gram of powder. After use, discard open packaging and unused product per facility waste protocol.

Indications: Vitále Collagen Powder Dressing is indicated for the management of minimally to heavily exuding acute and chronic wounds including:



Superficial or Deep

- Pressure Ulcers
- Venous Stasis Ulcers
- Diabetic Ulcers



Minimally to Heavy Exuding

- Surgical, dehisced surgical, and traumatic wounds
- Partial thickness burns.

Vitále Collagen Powder Dressings are also indicated for control of minor bleeding, including post-debridement.

Caution:

Vitále Collagen Powder Dressing is intended for topical use only. Avoid inhalation and flush immediately with water if product comes in contact with eyes.

Contraindications:

Vitále Collagen Powder Dressings should not be used on patients with known sensitivity or allergy to bovine collagen.

Application:

1. Clean the wound with sterile physiological saline using existing clinical protocol. Dry the surrounding skin gently.
2. Peel open Vitále packet and apply sufficient powder to insure approximately ¼” depth of coverage of the wound bed. Do not overpack or compress powder as it may expand as it absorbs wound exudate.
3. Choose and apply appropriate secondary dressing to secure and protect collagen powder, such as Vitále Cover or Composite Dressings, Vitále Hydrocolloid Dressings, etc.
4. Change dressings daily, as directed, when fully saturated or according to facility change policy, but at least every 7 days.
5. When changing, remove Secondary Dressing, and gently remove gel formed of dressing and exudate. Clean the wound with sterile physiological saline using existing clinical protocol. Dry the surrounding skin gently.

Federal Law (USA) restricts the sale and use of this device at the direction of authorized physicians and licensed healthcare practitioners.

– Product Line –


Catalog Number:	Description:	Size:	Dressings/Box:
22000	Vitále Collagen Powder Dressings	1 gm/packet	10

Made in USA for



CellEra, LLC Cincinnati, OH
www.celleralc.com

STERILE Sterile

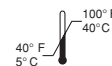

Storage Temperature

Single-Use

Keep Dry

Rx Only

Keep From Heat

Not Made With Natural Rubber Latex