

– Instructions for Use –

Product Description: Vitále® Composite Post-Op Dressings are absorbent, have an adhesive border, and provide an ideal cover for wounds of many types. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated for covering primary treatment dressings, and assist in protecting the wound, preventing bacterial contamination and maceration.

Composite Dressings have the same construction and add a waterproof yet vapor permeable film to provide protection. They can be used on areas of the body affected by incontinence, to keep the wound clean, or over treated wounds while showering or bathing. The breathable film allows excess gases to escape.

Vitále Composite Post-Op Dressings are available in a selection of convenient sizes. They are sterilized by radiation and must not be resterilized.

Indications: Vitále Composite Post-Op Dressings are for single use only and can be used to cover primary dressings and protect many types of wounds:



Superficial or Deep



Moderate to Heavy Exuding Wounds

- Post-Operative Wounds
- Skin Donor Site
- Second Degree Burns
- Post-Surgery Sites
- Pressure Sores
- Arterial Ulcers
- Venous Ulcers
- Diabetic Ulcers

Side Effects/Interactions: None have been observed to date.

Storage: Vitále Composite Post-Op Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

Application:



– Product Line –

Catalog Number:	Description:	Size:	Dressings/Box:
24106	Vitále Composite Post-Op Dressings	4"x6" (10cm x 15cm)	30
24110	Vitále Composite Post-Op Dressings	4"x10" (10cm x 25cm)	30
24114	Vitále Composite Post-Op Dressings	4"x14" (10cm x 35cm)	15



CellEra, LLC Cincinnati, OH
www.cellera.com



Do not use if package is damaged



Single-Use



Keep Dry



Storage Temperature



Sterilized using Gamma Radiation



Not made with natural rubber latex