

– Instructions for Use –

Product Description: Vitále® Cover & Composite Dressings are absorbent, have an adhesive border, and provide an ideal cover for wounds of many types. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated for covering primary treatment dressings, and assist in protecting the wound, preventing bacterial contamination and maceration.

Cover Dressings (also known as “bordered gauze”) have an absorbent gauze pad in the center, and a soft & strong backing made from spun-lace non-woven material. The adhesive border firmly holds the cover dressing in place, yet is easily removed at dressing change time.

Composite Dressings have the same construction and add a waterproof yet vapor permeable film to provide protection. They can be used on areas of the body affected by incontinence, to keep the wound clean, or over treated wounds while showering or bathing. The breathable film allows excess gases to escape.

Vitále Cover & Composite Dressings are available in a selection of convenient sizes. They are sterilized by radiation and must not be resterilized.

Indications: Vitále Cover & Composite Dressings are for single use only and can be used to cover primary dressings and protect many types of wounds:



Superficial or Deep




Moderate to Heavy Exuding Wounds

- Pressure Sores
- Arterial Ulcers
- Venous Ulcers
- Diabetic Ulcers
- Post-Operative Wounds
- Skin Donor Site
- Second Degree Burns
- Post-Surgery Sites


Side Effects/Interactions: None have been observed to date.

Storage: Vitále Cover & Composite Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.


Application:

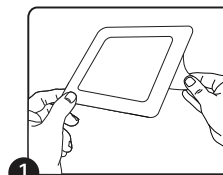

1 First remove one side of paper backing.

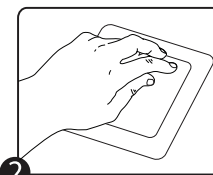

2 Place center of dressing over area and adhere one side.

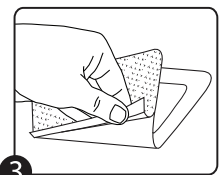

3 Remove remainder of backing and adhere.

– Product Line –

Catalog Number:	Description:	Size:	Dressings/Box:
24002	Vitále Composite Dressings	2" x 2" (5cm x 5cm)	30
24004	Vitále Composite Dressings	4" x 4" (10cm x 10cm)	30
24007	Vitale Composite Dressings	6" x 7" (15cm x 18cm)	30
24502	Vitále Cover Dressings	2" x 2" (5cm x 5cm)	30
24504	Vitále Cover Dressings	4" x 4" (10cm x 10cm)	30
24506	Vitále Cover Dressings	6" x 6" (15cm x 15cm)	30
24544	Vitále Cover Dressings	4" x 14" (10cm x 35cm)	20



CellEra, LLC Cincinnati, OH
www.cellera.com



Do not use if package is damaged



Single-Use



Keep Dry



Storage Temperature



Sterilized using Gamma Radiation



Not made with natural rubber latex