

VITÁLE®

Foam Dressings



Vitale® Adhesive Foam Dressings are 1-piece dressings for wounds with moderate to heavy drainage. The foam is highly absorbent, the adhesive gently but firmly holds the dressing in place and the film backing holds moisture in the dressing but allows vapor transmission. Available in 4"x4" and 6"x6" sizes.

Vitale Non-Adhesive Foam Dressings (4"x4") use the same absorbent foam and waterproof backing, and are designed to fill the wound and absorb exudate. May be cut to size. Cover with Vitale Cover Dressings.

We also have a 3x3 Non-Adhesive Foam Dressing that is slightly thicker than our standard foam.


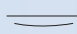



Vitale Heel Dressings are the one-size-fits-all answer to dressing heel wounds. The soft, conforming foam absorbs wound drainage, and the waterproof film backing allows vapor transmission. There is a light adhesive over the entire surface of the dressing to keep it in place during dressing change, and once applied comfortably the dressing should be covered with a soft elastic gauze to hold in place.



Indications:

Vitale Foam Dressings are for single use only and can be used for the treatment of the following wounds:

-  • Moderate to Heavy Exudate
-  • Superficial or
-  • Deep
- Pressure Ulcers
- Arterial Ulcers
- Venous Ulcers
- Skin Donor Sites
- Superficial & Deep Wounds
- Moderate to Heavy Exuding Wounds
- Diabetic Ulcers
- Second Degree Burns

Not indicated for dry wound conditions or for 3rd degree burns. Vitale Foam Dressings are sterilized by exposure to gamma radiation and must not be resterilized.

Features & Benefits:

- Highly absorbent
Absorbs many times its own weight, allowing extended wear time depending on the volume of exudate.
- Moist wound environment
The waterproof backing keeps exudates in the foam, and promotes a warm, moist healing environment.
- Breathable film backing
The film backing allows vapor transmission, to balance the fluid level as the foam fills with moisture.
- Waterproof backing
Keeps bacteria & contaminants out of the wound bed.

Ordering Information:

Product #:	Description:	Qty/box:	HCPC Codes:
20344	Vitale Adhesive Foam Dressings, 4"x4" (10cm x 10cm)	10	A6212
20366	Vitale Adhesive Foam Dressings, 6"x6" (15cm x 15cm)	10	A6212
20400	Vitale Foam Heel Dressings (one size fits all)	5	A6210
20444	Vitale Non-Adhesive Foam Dressings, 4"x4" (10cm x 10cm)	10	A6209
20463	Vitale Non-Adhesive Foam Dressings, , Extra Thick 3"x3" (7.5cm x 7.5cm)	10	A6209

