

VITÁLE® Non-Adherent Oil Emulsion Dressings

– Instructions for Use –

Product Description: Vitále® Non-Adherent Oil Emulsion Dressings are semi-occlusive primary dressings made of gauze mesh impregnated with an oil emulsion blend. They maintain a moist wound environment to support healing while allowing exudates to pass through. They cling & conform to body contours. Use beneath a secondary dressing, such as Vitále® Cover Dressings or Vitále Hydrocolloid Dressings.

The oil emulsion blend helps prevent dressing adherence, while forming a semi-occlusive barrier, allowing excess exudate in the wound to pass through to be absorbed by the secondary dressing. The dressings may be cut or folded to fit the wound cavity or wrapped around extremities. Vitále Oil Emulsion Dressings are sterilized by exposure to gamma radiation and must not be resterilized.

Indications:

Vitále Non-Adherent Oil Emulsion Dressings can be used as a primary dressing for wounds which are:



Superficial to Full Thickness

- Incision Sites
- Donor Sites & Skin Graft sites
- Abrasions & Minor Burns
- Venous & Pressure Ulcers
- Staples, Suture lines & Lacerations



Dry to Highly Exuding

Caution:

- Do not use in cases of known sensitivity to the dressing & its components.
- Wounds should be reviewed and appropriately managed, under the direction of a medical practitioner.

Side Effects & Interactions:

None have been observed to date.

Storage:

Vitále Oil Emulsion Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

Application:

1. Clean and rinse the wound, using appropriate Wound Cleanser, such as Vitále Wound Cleanser Spray.
2. Skin around the wound should be clean & dry. Use a Skin Barrier Prep, such as Vitále Skin Protectant Wipes, as needed.
3. Using aseptic techniques, remove dressing from package and apply to moist wound bed. Dressing may be cut to size. Overlap wound edges.
4. Cover with a Secondary Absorbent Dressing, such as Vitále Cover or Composite Dressings or Vitále Hydrocolloid Dressings.
5. Dressing change as directed, when strike through occurs, or at least every 7 days


– Product Line –

Catalog Number:	Description:	Size:	Strips/Box:
58303	Vitále Oil Emulsion Dressings	3"x 3" (7.5 cm x 7.5 cm)	50
58308	Vitále Oil Emulsion Dressings	3" x 8" (7.5 cm x 20 cm)	24
58316	Vitále Oil Emulsion Dressings	3" x 16" (7.5 cm x 40 cm)	36
58509	Vitále Oil Emulsion Dressings	5" x 9" (12.5 cm x 22.5 cm)	12



CellEra, LLC Cincinnati, OH
www.celleralc.com

STERILE R Sterile

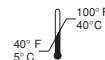

Storage Temperature

Single-Use

Keep Dry