

– Instructions for Use –

Product Description: Vitále® Silicone Post-Op Composite Dressings are absorbent, have a non-traumatic silicone adhesive border, and provide an ideal cover for wounds created by surgery. They are soft and flexible dressings that are very comfortable on fragile skin, and have a long wear time. These dressings are indicated as a secondary dressing to cover primary treatments & dressings, or as a primary dressing over post surgical incisions. They assist in protecting the wound during healing and preventing bacterial contamination or maceration.

Vitále Silicone Post-Op Composite Dressings have an absorbent non-adherent pad in the center, held firmly in place by the silicone, uniquely formed in an oval shape open to the passage of fluids into the pad. The silicone adhesive allows easy & safe removal of the dressing at change time, without trauma to the patient. The soft and strong backing is made from flexible polyurethane that provides a waterproof barrier. This breathable film allows gases, including water vapor, to escape the dressing.

Vitále Silicone Post-Op Composite Dressings are available in three convenient sizes. They are sterilized by ethylene oxide and must not be resterilized.

Indications: Vitále Silicone Post-Op Composite Dressings are for single use only and can be used as primary or secondary dressings, to protect many kinds of wounds:



Superficial or Deep




Minimal to Heavy Exuding Wounds

- Post-Operative Wounds
- Skin Donor Sites
- Second Degree Burns
- Post-Surgery Sites
- Minor Wounds and Lacerations
- Pressure Sores
- Arterial Ulcers
- Venous Ulcers
- Diabetic Ulcers


Side Effects/Interactions: None have been observed to date.

Storage: Vitále Silicone Post-Op Composite Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.


Application:

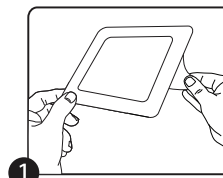

1 First remove one side of the backing.

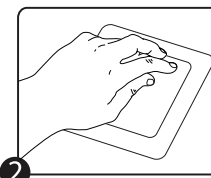

2 Place center of dressing over area and adhere one side.

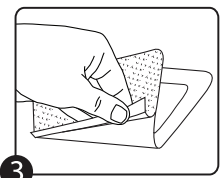

3 Remove remainder of backing and adhere.

– Product Line –

Catalog Number:	Description:	Dressing Size:	Dressings/Box:
24406	Vitále Silicone Post-Op Composite Dressings	4"x6" (10cm x 15cm)	30
24410	Vitále Silicone Post-Op Composite Dressings	4"x10" (10cm x 25cm)	30
24414	Vitále Silicone Post-Op Composite Dressings	4"x14" (10cm x 35cm)	15

STERILE EO Sterilized by exposure to Ethylene Oxide

Single-Use

Storage Temperature

Not Made with Natural Rubber Latex

Keep Dry

