

– Instructions for Use –

Product Description: Vitále® Petrolatum Dressings are non-adherent occlusive strip dressings made of cotton gauze impregnated with white petrolatum. They maintain a moist wound environment to support healing. They cling & conform to body contours. Use beneath a secondary dressing, such as Vitále® Cover Dressings or Vitále Hydrocolloid Dressings.

Petrolatum forms an occlusive barrier, holding moisture in the wound and protecting from environmental contamination. The Dressings may be cut or folded to fit the wound cavity or wrapped around extremities. Vitále Petrolatum Dressings are sterilized by exposure to gamma radiation and must not be resterilized.

Indications:

Vitále Petrolatum Dressings can be used for management of wounds which are:

 **Superficial to Partial Thickness**

- Incision Sites
- Donor Sites & Skin Graft sites
- Abrasions & Minor Burns
- A Primary Occlusive Layer

 **Dry to Light Exuding**

Caution:

- Do not use in cases of known sensitivity to petrolatum.
- Wounds should be reviewed and appropriately managed, under the direction of a medical practitioner.

Contraindications:

Vitále Petrolatum Dressings form an occlusive dressing, & as such are not indicated for highly exuding wounds.

Side Effects & Interactions:

None have been observed to date.

Storage:

Vitále Petrolatum Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

Application:

1. Clean and rinse the wound, using appropriate Wound Cleanser, such as Vitále Wound Cleanser Spray.
2. Skin around the wound should be clean & dry. Use a Skin Barrier Prep, such as Vitále Skin Protectant Wipes, as needed.
3. Using aseptic techniques, remove dressing from package and apply to moist wound bed. Dressing may be cut to size. Overlap wound edges.
4. Cover with a Secondary Absorbent Dressing, such as Vitále Cover or Composite Dressings or Vitále Hydrocolloid Dressings.
5. Dressing change as directed, when strike through occurs, or at least every 7 days.


– Product Line –

Catalog Number:	Description:	Size:	Strips/Box:
56108	Vitále Petrolatum Dressings	1" x 8" (2.5 cm x 20 cm)	50
56309	Vitále Petrolatum Dressings	3" x 9" (7.5 cm x 22.5 cm)	50
56318	Vitále Petrolatum Dressings	3" x 18" (7.5 cm x 40 cm)	12
56636	Vitále Petrolatum Dressings	6" x 36" (15 cm x 91 cm)	12



CellEra, LLC Cincinnati, OH
www.celleralc.com

STERILE Sterile

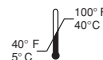

Storage Temperature

 Single-Use

 Keep Dry